


VERDICT[®]-II

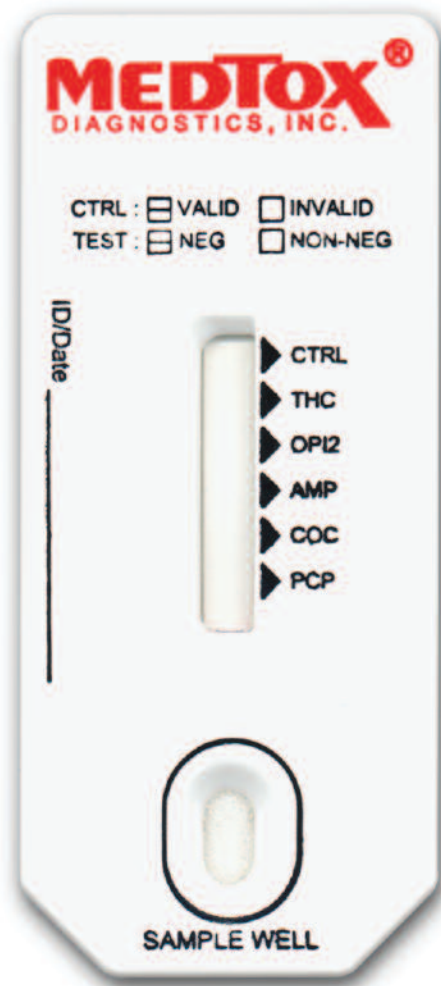

Training and Certification Program

Presented by

MEDTOX[®]

Intended Use

VERDICT-II is a one-step immunochromatographic test for the rapid qualitative detection of cannabinoids (THC), cocaine, opiates, amphetamines, and phencyclidine (PCP), methamphetamines, methadone, tricyclic antidepressants, barbiturates, and benzodiazepines in human urine. The test detects the major metabolites of these drugs at the cutoff levels. It is not for over-the-counter sale.

The VERDICT-II Test Device provides only a preliminary analytical test result. A more specific alternate chemical method must be used in order to obtain a confirmed analytical result. Laboratory confirmation with gas chromatography/mass spectrometry (GC/MS) is the preferred confirmatory method. As always, clinical consideration and professional judgment should be applied to any drugs-of-abuse test result.

Precautions

- VERDICT-II Test Devices should be stored at 35-77°F (2-25°C) and brought to room temperature prior to use. Store kits refrigerated if room or ambient temperatures could exceed 77°F.
- Dilution/adulteration may produce inaccurate test results.
- Do not use test after Expiration Date on package label is exceeded.
- Do not freeze test kits.
- VERDICT-II provides only a preliminary analytical test result. A more specific alternate chemical method must be used in order to obtain a confirmed analytical result. (GC/MS is the preferred confirmatory method.)
- Urine specimens and materials coming into contact with urine should be handled and disposed of appropriately (see Package Insert).
- Use a new pipette for each urine sample.
- VERDICT-II Test Devices should remain in original sealed foil package until ready to use. If pouch integrity is damaged, do not use.
- The VERDICT-II Test Device is for in vitro diagnostic use only.

Test Procedures

Step 1 Open one foil sealed device for each test performed and print date (on Date line) and patient or sample identification (on ID line) on VERDICT-II Test Device.

Step 2 Apply Sample as Follows:

- Hold the 100 µl sample pipette by the upper bulb.
- Lower the pipette stem into the urine sample.
- Squeeze the upper bulb then release it slowly. This motion will draw 100 µl of urine into the stem. The urine sample should reach the top of the stem and overflow slightly into the middle bulb, if not, repeat this process.
- Dispense the 100 µl sample into the sample well by squeezing the upper bulb. This will empty the stem delivery 100 µl of sample.
- Apply 100 µl of sample ONE time into each sample well.

Step 3 Read test results at 5-15 minutes after sample application.

Interpreting Test Results


If you are unsure of the test result, send the specimen to MEDTOX for confirmation. Any line of faint color intensity visible within 15 minutes indicates a negative test result. **DO NOT READ TEST AFTER 15 MINUTES.**

WHEN IN DOUBT—SEND IT OUT



INVALID


The absence of a reddish-purple control (CTRL) line indicates the test is invalid. The urine sample should be retested on a new VERDICT-II Test Device

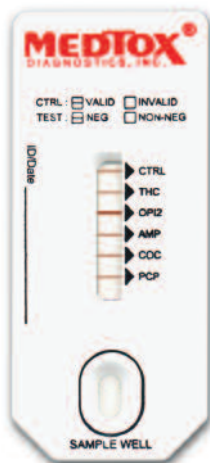

NEGATIVE

The appearance of both a reddish-purple control (CTRL) line and a specific drug line indicates a negative test result.

The color intensities of the control line and a specific drug line may not be equal; any line of faint color intensity visible within 15 minutes indicates a negative test result.

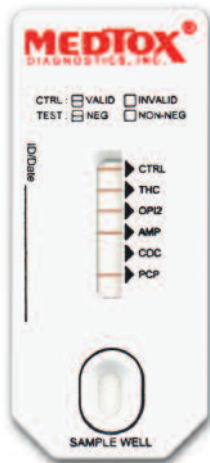

NON-NEGATIVE

The appearance of a reddish-purple control (CTRL) line and the absence of a line next to a specific drug name indicate a preliminary positive test result for that drug.

Limitations of Procedure

- VERDICT-II is only for use with unadulterated human urine.
- A non-negative result for any drug(s) in VERDICT-II does not indicate or measure intoxication, it only indicates the presence of specific drugs in the urine specimen.
- Test results interpreted after 15 minutes may not be consistent with the original result obtained at the 5-minute reading.
- Certain medications containing interfering substances may produce a positive result. Refer to product insert for further information.
- Some result lines will be fainter than others. Any indication of a line represents a negative result.


Training Program Completed

You have now completed the VERDICT-II training program.

To achieve certification as a tester with this device, you must successfully complete the following 10-question certification quiz with a score of 80% or higher.

If you have any questions and would like to speak to a service representative, please call us at 1-888-557-2590 or email: VerdictTraining@medtox.com.

VERDICT® II

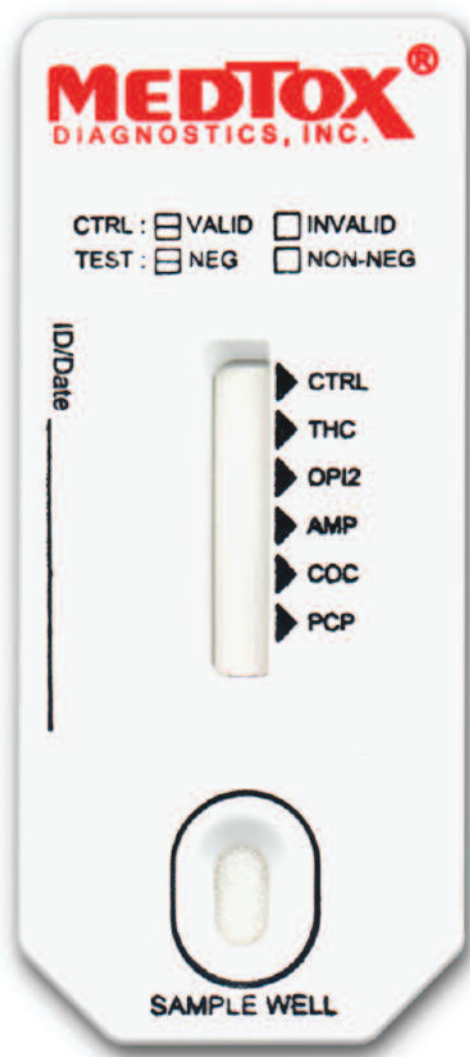

Certification Quiz

Presented by

MEDTOX®

VERDICT-II Certification Quiz


READING EACH QUESTION COMPLETELY AND CHOOSE THE BEST ANSWER.

- 1) VERDICT-II provides a:
 - A) confirmed analytical result
 - B) quantitative analytical result
 - C) preliminary test result
 - D) qualitative test result
 - E) C and D only

- 2) The entire test is invalid if:
 - A) no control (CTRL) line forms in the window
 - B) the urine sample smells bad
 - C) control (CTRL) line only forms
 - D) A and C only


- 3) VERDICT-II devices should be stored at:
 - A) 25-57°F
 - B) 35-77°F
 - C) 2-25°C
 - D) A and B only
 - E) B and C only

- 4) Interpret the following test:
 - A) negative for all 5 drugs
 - B) valid test
 - C) invalid test
 - D) non-negative for THC
 - E) A and B only

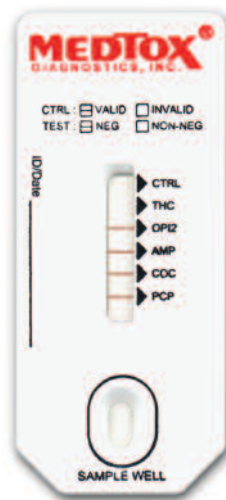


5) Interpret the following test:

- A) non-negative for amphetamines
- B) negative for all 5 drugs
- C) valid test
- D) invalid test
- E) A and C only

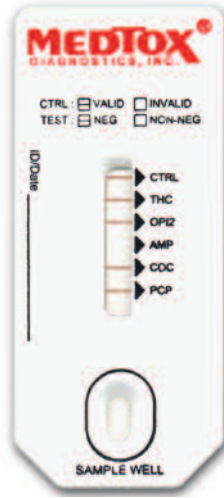


6) Interpret the following test:

- A) non-negative for THC, PCP
- B) negative for THC, PCP
- C) negative for all 5 drugs
- D) non-negative for OPI,AMP, COC
- E) invalid test

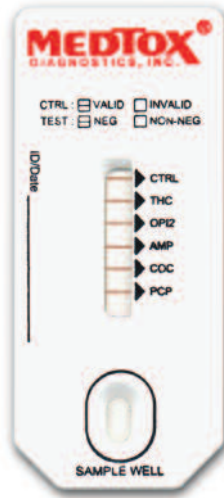

7) Interpret the following test:

- A) non-negative for THC, COC
- B) non-negative for OPI,AMP,PCP
- C) negative for OPI only
- D) negative for THC, COC
- E) invalid test

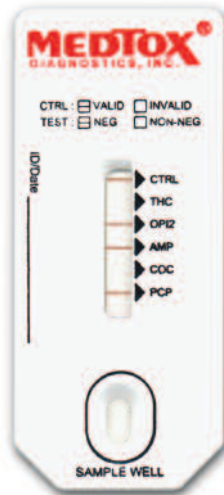

- 8) The control line (CTRL):
- A) should exhibit a reddish-purple line
 - B) should have no line appear
 - C) is used as a comparative indicator
 - D) B and C only
- 9) The test kit may be used after the expiration date:
- A) if the kit has not been opened
 - B) never
 - C) within one month after the expiration date
 - D) only if the kit has been frozen
- 10) The following precautions must be taken when using the VERDICT-II device:
- A) do not freeze the device
 - B) use a new pipette for each urine sample
 - C) read the results within five to 15 minutes
 - D) all of the above

VERDICT II

Certification Quiz Answer Sheet

Enter your answers in the spaces below, fill in your name and other requested information, and fax your answer sheet to MEDTOX. A Certificate of Achievement will be sent to you if you receive a score of 80% or higher.

Question 1: _____ Question 6: _____
Question 2: _____ Question 7: _____
Question 3: _____ Question 8: _____
Question 4: _____ Question 9: _____
Question 5: _____ Question 10: _____

Name (please print): _____

Company Name: _____

Address: _____

City, State, Zip: _____

Telephone: _____

Fax: _____

Fax to 651-286-6297

Or mail to: MEDTOX Laboratories
402 West County Road D
Saint Paul, MN 55112
Attn: Verdict II Certification

Or e-mail answers to: VerdictTraining@medtox.com

9/05

© Copyright 2005, MEDTOX Scientific, Inc.
MEDTOX and VERDICT are registered trademarks