


MEDTOX[®] SURE-SCREEN[®]

Instructions for Use



The device detects one or more of the following drug classes at the following concentrations:

Amphetamine (AMP)	300 ng/ml
Barbiturates (BAR)	200 ng/ml
Buprenorphine (BUP)	10 ng/mL
Benzodiazepines (BZO)	200 ng/ml
Cocaine (COC)	100 ng/ml
Methadone (MTD)	200 ng/ml
Methamphetamine (mAMP)	1000 ng/ml
Opiates (OPI)	100 ng/ml
Oxycodone (OXY)	100 ng/ml
Phencyclidine (PCP)	25 ng/ml
Propoxyphene (PPX)	300 ng/ml
Cannabinoids (THC)	40 ng/ml

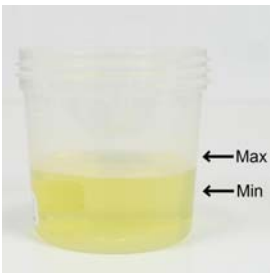

Donor voids into cup. The urine volume should be between the minimum and maximum lines. Screw lid securely onto cup.



Remove SURE-SCREEN[®] device from foil pouch. Hold the cup and align device and cup as illustrated.



Place the device securely on to the lid--the parts should interlock easily and move freely clockwise.



Rotate the device clockwise ¼ of a turn (i.e. from 9 o'clock to 12 o'clock) until it snaps into place.



Place cup on its side as shown above to start urine flow. (If less than 45 ml of urine, you may need to tilt cup forward to begin flow).

C = Control
T = Test



A line must appear in the "C" area in order for the test to be valid.

In the example above, the Control line is missing for the Benzodiazepines test and should be interpreted as an invalid test result for that drug.



Any line in the test area is to be interpreted as negative test result.

The absence of a line in the test area is to be interpreted as a non-negative.

In the example above, the Opiates test is non-negative (absence of a test line).

NOTE:
Line intensity may vary. Any reddish-purple line visible at 5 minutes (even a faint or broken line) indicates a negative result.

Drug Screen
Read Time: 5 minutes.

FOR FORENSIC USE ONLY.

For complete product information refer to the PACKAGE INSERT available at <http://www.medtox.com/ProductTraining.aspx> or contact Technical Support at 1-877-643-5703. SURE-SCREEN[®] 12-panel ForensicUseOnly Package INSERT, P/N 102109 Rev. 5/09

Note: If LFAS adulterant strip is present, refer to the Color Comparator Chart to interpret results.

MEDTOX Diagnostics, Inc. • 1238 Anthony Road • Burlington, NC 27215
Phone: 1-877-457-0613 • Fax: 651-286-6222 • E-mail: sales@medtox.com