



EZ-SCREEN[®] Cup

Quick Reference Guide


MEDTOX Diagnostics, Inc. • 1238 Anthony Road
 Burlington, NC 27215
 Phone: 800-832-3244 • Fax: 651-286-6222
 E-mail: sales@medtox.com

Drug tests for *in vitro* diagnostic use only
 Adulterant tests are for forensic use only

The device detects one or more of the following drug classes at or above the following cutoff concentrations:

Amphetamine (AM)	1000 ng/ml	Opiates (OP)	2000 ng/ml
Barbiturates (BA)	200 ng/ml	Oxycodone (OX)	100 ng/ml
Benzodiazepines (BZ)	300 ng/ml	Phencyclidine (PC)	25 ng/ml
Cocaine (CO)	300 ng/ml	Propoxyphene (PP)	300 ng/ml
Methamphetamine (mA)	1000 ng/ml	Cannabinoids (TH)	50 ng/ml
Methadone (MT)	300 ng/ml		

Instructions

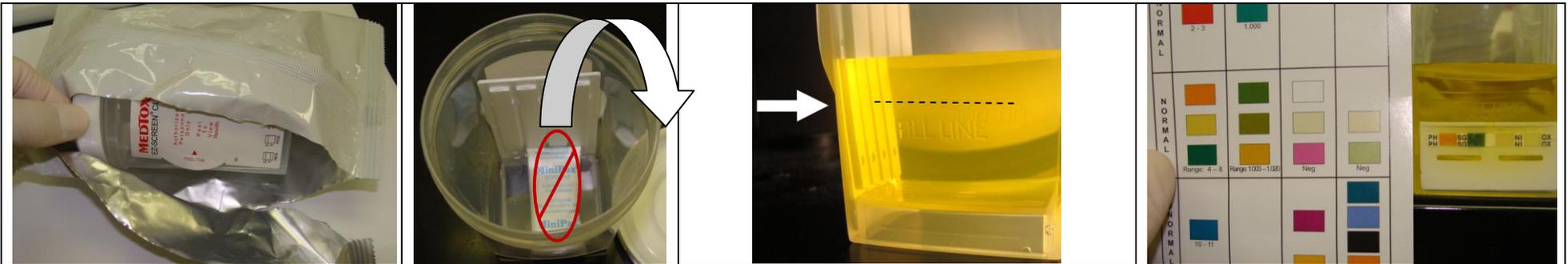

- Open pouch and remove device. Note: product must be allowed to come to room temperature (18-25°C / 64-77°F) if it is stored cooler than that.
 •Acceptable Storage Temp: (2-25°C / 36-77°F)
 •Do not use expired product.
- After opening pouch, remove lid and **DISCARD desiccant**.
- Donor voids into cup. The **Urine volume** should be **at or above** the Fill Line.
 •Always keep container **upright and minimize handling** during test development.
 •**TIGHTEN lid securely onto the cup** to avoid potential loss/leaking of sample if shipment to lab is required.
- For panels with Adulterant Strip only -- Turn the cup and read the Adulterant strip by comparing it to the color chart provided.
 •Tests begin developing upon voiding into the cup.
 •A timing device (like a wall clock, kitchen timer, watch or cell phone) is recommended.

READING THE TEST RESULTS:		INTERPRETING RESULTS:	VALID: A reddish-purple line appears in the "C" area for each test strip.	
5. Remove privacy tab to read results. Read Time: 5 minutes after voiding into cup	•If needed, tilt cup back slightly so you can see lines clearly. NOTE: Line intensity may vary. The presence of any reddish-purple line, whether faint or broken, is to be interpreted as a negative result.	☆ INVALID: Above Test is Invalid-- No control line "C" present. Retest donor with a new device if you get an invalid result.	NEGATIVE: Any reddish-purple line (even a faint or broken line) visible at 5 minutes in the test area (along with a valid control) is a negative result. Example above: all five test results are negative, and test is Valid.	PRESUMPTIVE POSITIVE: The absence of a line in the test area (along with a valid control) is a presumptive positive result. Example above: the OP (Opiate) test is presumptive positive and the test is Valid.

For complete product information refer to the EZ-SCREEN[®] CUP Package Insert-II, Rev. 8/11 available at <http://www.medtox.com/ProductTraining.aspx> or contact Technical Support at 1-877-643-5703.