


MEDTOX[®] SURE-SCREEN[®]

Instructions for Use

- C = Control Line
 T1 = Test Line for drug name on top portion
 T2 = Test Line for drug name below (if no print on lower color, then no T2 drug tested)

Bring pouched device to room temp (18-25°C / 64-77°F) before opening.

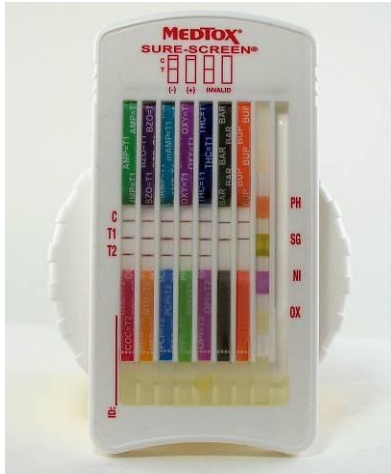

The device detects one or more of the following drug classes at the following concentrations:

Amphetamine (AMP)	300 ng/ml
Barbiturates (BAR)	200 ng/ml
Benzodiazepines (BZO)	200 ng/ml
Cocaine (COC)	100 ng/ml
Methadone (MTD)	200 ng/ml
Methamphetamine (mAMP)	1000 ng/ml
Opiates (OPI)	100 ng/ml
Oxycodone (OXY)	100 ng/ml
Phencyclidine (PCP)	25 ng/ml
Propoxyphene (PPX)	300 ng/ml
Cannabinoids (THC)	40 ng/ml

Note: If LFAS adulterant strip is present (to the right of drug test strips), refer to the Color Comparator Chart to interpret results.

Obtain urine. Urine level must be between the Min / Max lines on cup. If urine was refrigerated, bring to room temp and mix well before testing.

Screw lid onto cup -- **TIGHTEN FIRMLY.** Tear open pouch and remove device. Align device and cup as shown.

Place the device securely on to the lid---the parts should interlock easily and move freely clockwise.

Rotate the device clockwise ¼ of a turn (i.e. from 9 o'clock to 12 o'clock) until it snaps into place.

Place cup on its side as shown above to start urine flow.

READING THE TEST RESULTS

Read Time: 5 minutes

For examples at right:
 C = Control
 T1 Tests = AMP, THC, BZO
 T2 Tests = COC, OPI

VALID:
 A reddish-purple line appears in the "C" area for each test.

INVALID:
 No control line "C" present. Retest urine with a new device.

NEGATIVE:
 Any reddish-purple line visible at 5 min. in the test area is a negative result (even a faint or broken line).
 Example above: all 5 test results are negative.

PRELIMINARY POSITIVE:
 The absence of a line in the test area is a preliminary positive result.
 Example above: the AMP (Amphetamine) test is a preliminary positive.

Example Sufficient URINE Present --about ¼ inch *evenly distributed*-- is all that is needed for the device to migrate properly. More urine is NOT necessary! Allow test device to rest undisturbed for 5 minutes.

Strips with two tests will be labeled with two colors. The top color will indicate the T1 test with drug test name=T1. The bottom color will indicate the T2 test with test name=T2. Strips with only one color will have test results appear at the T1 position only.

Seal Properly if Transporting to Laboratory for confirmation:

- Tighten the lid
- Use dual seal provided with cup
 - Seal sample port
 - Seal vent hole

For complete product information refer to the PACKAGE INSERT available at <http://www.medtox.com/ProductTraining.aspx> or contact Technical Support at 1-877-643-5703.

SURE-SCREEN[®] Package INSERT Rev. 4/10, P/N 101490.

**MEDTOX Diagnostics, Inc. • 1238 Anthony Road
 Burlington, NC 27215
 Phone: 1-877-457-0613 • Fax: 651-286-6222
 E-mail: sales@medtox.com**